



Product name: Skin Marker For Medical Use

Model types: single tip and double tip

Specifications:


Serial No.	Accessories
660110	Regular tip (1mm) , without ruler
660111	Regular tip (1mm) , with ruler
660120	Fine tip (0.5mm) , without ruler
660121	Fine tip (0.5mm) ,with ruler
660130	Double tip (1mm, 0.5mm) without ruler
660131	Double tip (1mm, 0.5mm) with ruler

Product Properties :

Draw on skin smoothly, non-toxic, harmless and non-stimulating to skin.

Components :

Core, Tip, Body, Cap, Cover, Ink, Ruler (option)

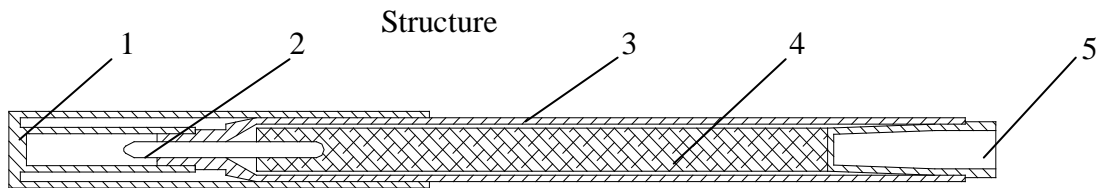

1. Cap, 2. Tip 3. Body, 4. Core, 5. Cover

Application: Positioning identifications in Ordinary Surgery, Enterochirurgia, Orthopedics, Neurohormone Surgery, Cardiovascular Surgery and Radiotherapy.

Direction :

1. Clean skin, sterilize with Alcohol and make it dry;
2. Make marking with Skin Marker (the Ink is a kind of sterilize agent), further sterilize is not required or
3. Sterilize again with Iodine, (this would not remove the mark made)
4. Marking can be removed by Alcohol or Acid.

Cautions:

1. Obtain professional medical advise before use.
2. Consider whether the patient is sensitive to gentian violet;
3. Apply to single patient only;
4. Do not use if package is damaged.

Prohibited :

Use on mucous membrane is prohibited,
Use on damaged skin or wounds is not recommended.

Possible complications :

No record.

Storage :

- Store in cool, ventilated and dry place, avoid contact with corrosive gas.
- Temperature: $\leq 40^{\circ}\text{C}$
- Humidity: 30~80%
- Validity 2 years from manufacture.

Sign instructions:

Attention, make a reference to the attached file



Don't use again



Sterilized by Irradiation



Batch code



Commodity code



Expiry date



Manufacturer name



Manufacturing date

Medical Device Manufacturing Permit No. : Guangdong Food and Drug Administration No. 2010005

Medical device Registration No. : Guangdong Food and Drug Administration No.2011 of 1010001

Product Standard No.: YZB/001-2011

Manufacturer : Dong Guan Concord Medical Product Co., Ltd.

Address : 4th Floor, Block C, Yue Neng Industrial Park, Xia Bian Village,
Hou Jie Town, Dong Guan City, Guang Dong Province, China

Zip code : 523960

Telephone No. : +86-769-88919066

Fax number : +86-769-88919050

After sales service : Dong Guan Concord Medical Product Co., Ltd.

EU Rep : Wellkang Ltd.

: 29 Harley St., London W1G 9QR UK (www.ce-marking.eu)

Hong Kong Office : Concord Medical Limited

Rm 2509 Trend Centre, 29-31 Cheung Lee St., Chaiwan, HK.

Telephone No. : +852-2866 6308

Fax No. : +852-3542 5676

E-mail : info@concord-medical.com Website : www.concord-medical.com