
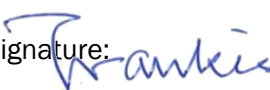
Document Name and Number:

D-730-02 Sterile Disposable Skin Marker IFU (Radiation Sterilization)_V2


Document information, authorship, and approvals			
Author signs to confirm technical content			
Prepared by: Denise Mak	Job title: Quality Assistant	Signature: 	Date: 7/10/2024
Subject matter expert reviewer signs to confirm technical content			
Reviewed by: Frankie Ng	Job title: Managing Director	Signature: 	Date: 7/10/2024

Document history			
Revision No.	Revision Date	Description of change	Author
V1	7/10/2024	Initial Release	Denise Mak
V2	25/11/2025	Addition of Dual Tips	Denise Mak

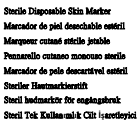EC REP mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen

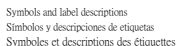Document No. D-730-02
Version 2nd Release
Issue Date 25/11/2024

	Item Number número de artículo numéro d'article codice articolo número de item Katalognummer Artikelnummer Ūrtn numarast
	lot number número de lote numéro de lot número di lotto número de lote Chargennummer Partinummer Lot Numarast
	Date of manufacture Fecha de manufactura Date de fabrication Data di produzione Data de fabricație Herstellungsdatum Tilverkingsdatum Ūretim tarihi
	Use by date Utilizator por fecha Utilisier por date Usare entro la data Usar por data Verwendbar bis Använd senast datum Tarihi Güte Kulları
	Keep away from sunlight Mantener alejado de la luz solar Tenir à l'écart du soleil Tenere lontano dalla luce solare Mantenha longe da luz solar Vor Sonnenlicht schützen Håll borta från solljus Güneşten uzak tutun
	Do not use if packaging is damaged and consult instructions for use No utilizar si el paquete está dañado y consultar las instrucciones de uso. Ne pas utiliser si l'emballage est endommagé et consulter les instructions d'utilisation No utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso Nao use se e embalagem estiver danificada e consulte as instruções de uso Niell verivenden, wenn die Verpackung beschädigt ist, und Gebrauchsanweisung beachten Använd inte om förpackningen är skadad och se bruksanvisningen Paket hasırlayış kullananıvın ve kullatmı talimatlarına bakın.

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ENGLISH Sterile Disposable Skin Marker

Description

Skin marker is a medical pen used for marking on the patient's skin before or during surgical procedures



Specifications

#	Models	With Ruler	containing 6 labels	containing 9 labels
1.	Regular Tip	No	No	No
2.	Regular Tip	Yes	No	No
3.	Regular Tip	Yes	Yes	No
4.	Regular Tip	Yes	No	Yes
5.	Fine Tip	No	No	No
6.	Fine Tip	Yes	No	No
7.	Fine Tip	Yes	Yes	No
8.	Fine Tip	Yes	No	Yes
9.	Dual Tips	No	No	No
10.	Dual Tips	Yes	No	No
11.	Dual Tips	Yes	Yes	No
12.	Dual Tips	Yes	No	Yes

Characteristics

Skin marker is used in marking and selecting the surgical sites before or during surgical procedures. Long-lasting scrub-resistant, practically odourless ink, contains primarily gentian violet, which is non-toxic, non-allergenic and non-irritating. Single-use, radiation sterilized product that will not come off easily during the disinfection procedure with 75% alcohol and iodophors. Quick, easy, and convenient to use in general surgery, plastic surgery, orthopaedic surgery, neurosurgery, cardiovascular surgery, and radiation therapy.

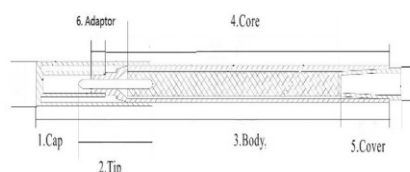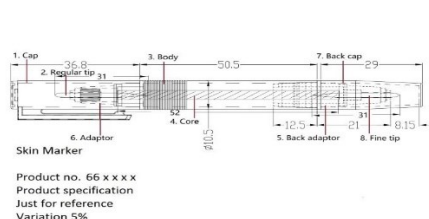Product construction

Intended use

Skin marker and selection of the surgical site for diagnostic and interventional radiology, surgery, medical teaching and treatment of skin diseases.

Directions for use

1. Sterilize the intact skin with 75% alcohol and iodophors according to standard procedure and let it air dry (about 6 seconds).
2. Test the marker on a small area of intact skin to ensure it is working properly and to check for any allergic reactions.
3. Apply gentle pressure while marking to ensure the dye transfers effectively and the mark is visible.

4. If additional marks are needed, repeat the previous steps as necessary.
5. After use, dispose of the marker safely and properly as medical waste.
6. Marking can be easily removed with a Chlorhexidine Gluconate solution.

Warnings

1. Consider the possibility that the patient may be allergic to Gentian Violet. If patient feels uncomfortable while using, stop using and treat appropriately.
2. Radiation sterilized, for single use only, do not reuse or re-sterilize. The reuse may cause cross contamination which leads to microbial infection.
3. Should be used on dry skin. If using on sweating or wet skin, please wipe dry and operate as above Directions for use.
4. Protect the tip during use and cover with cap when finish using.
5. It should be used immediately after opening the package and disposed of safely and properly as medical rubbish after use.
6. Do not use if package or product has been damaged or soiled.
7. DO NOT ingest. In case of accidental ingestion, DO NOT induce vomiting. Rinse mouth thoroughly with water and seek for medical advice.
8. Avoid contact with eyes. In case of accidental contact, flush eyes with water and seek medical advice.

Contraindication

Use on mucosa and open wound is not recommended.

Possible complications

None

Storage

Store in a cool, dry place. Avoid exposure to water, direct sunlight extreme temperature, high humidity or corrosive gases.

Temperature limit 5C° - 25C°

Sterile for 3 years. Expiry date : see package

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