



Product Manuel

Skin Marker For Medical Purpose




Product No.	Specification	Product No.	Specification
CC 663020	Regular tip 1 mm	CC 663120	Fine tip 0.5mm
CC 663021	Regular tip 1 mm sterile	CC 663121	Fine tip 0.5mm sterile
CC 663022	Regular tip 1 mm with ruler sterile	CC 663122	Fine tip 0.5 mm with ruler sterile

Application : Positioning identification in ordinary surgery, Enterochirurgia, Orthopedics, Necrohormone Cardiovascular Surgery and Radio therapy

Property : Draw on skin smoothly, harmless and non-stimulating to skin

Direction :

1. Clean skin, sterilize with Alcohol and make it dried
2. Make marking with Skin Marker, further sterile may not required or
3. Sterilize again with Iodine, (this would not remove the mark made)
4. Marking can remove easily by Chlorhexidine Gluconate.

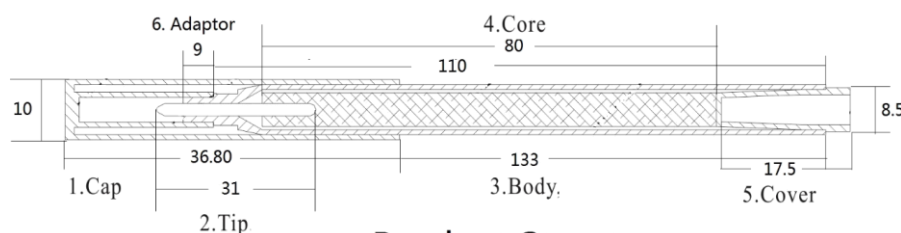

Product Structure

Cautions: Obtain professional medical advise before use
Consider whether the patient is sensitive to gentian violet
Apply to single patient only
Do not use if package is damaged

Prohibited : Use on mucous membrane and damaged skin or wounds is not recommended.

Possible complications : No record

Storage : Store in cool, ventilated and dry place, avoid contact with corrosives.
Temperature: $\leq 40^{\circ}\text{C}$; Humidity: 30~80%; Validity 3 years from date manufacture.



EN ISO 13485:2012/AC:2012

No. SX 60099840 0001

CE 0197 No. HD 60099839 0001

SFDA Cert No. Guangdong 2011 no. 1010001


Sign instructions:



Make reference to attached file



Disposable

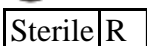

Sterile radiation



Product number



Batch number



Expiry date



Date of manufacture



Name of manufacturer

EC Rep : Welkeng Ltd (www.ce-marking.eu)

29 Harley St., London W1G 9QR, UK

Mftr : Dongguan Concord Medical Products Co, Ltd.

Address : 4th Floor, Blk C, Yue Neng Industrial Park,

Xia Bian Village, Hou Jie, Dong Guan, China

Zip code: 532 960

Tel no. : 86-769-8891 9066

Fax no. : 86-769-8891 9050

Service: Dongguan Concord Medical Product Co, Ltd.