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
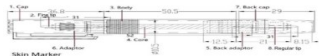
产品结构图 / Primary structure

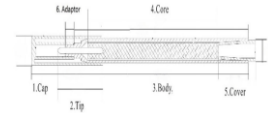Skin Marker  
Product no. 66 x x x x  
Product specification  
Just for reference  
Variation 5%

**东莞康钧医疗用品有限公司** 邮编: 523960  
地址: 东莞市厚街下汴村富民路粤能工业城C栋四楼  
Dongguan Concord Medical Products Ltd.  
Add: Floor 4, Block C, Yue Neng Ind Park  
Xia Bian Village, Houjie, Dongguan, China  
Tel : +86-769-88919066  
Fax : +86-769-88919050  
<https://www.concord-medical.com>

**EC REP** mdi Europa GmbH  
Langenhagener Str. 71  
30855 Langenhagen

Document No. D-730-40  
Version 1st Release  
Issue Date 2024/12/09

**[ENGLISH]** Sterile Disposable Skin Marker

**Description**

Skin marker is a medical pen used for marking on the patient's skin before or during surgical procedures.

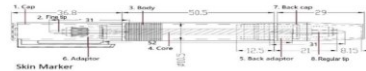
**Specifications**

#	Models	With ruler	Containing 6 labels	Containing 9 labels
1	Regular Tip	No	No	No
2	Regular Tip	Yes	No	No
3	Regular Tip	Yes	Yes	No
4	Regular Tip	Yes	No	Yes
5	Fine Tip	No	No	No
6	Fine Tip	Yes	No	No
7	Fine Tip	Yes	Yes	No
8	Fine Tip	Yes	No	Yes
9	Dual Tip	No	No	No
10	Dual Tip	Yes	No	No
11	Dual Tip	Yes	Yes	No
12	Dual Tip	Yes	No	Yes


**Characteristics**

Skin marker is used in marking and selecting the surgical sites before or during surgical procedures.  
Long-lasting scrub-resistant, practically odourless ink, contains primarily gentian violet.  
Single-use, radiation sterilized product that will not come off easily during the disinfection procedure with 75% alcohol and iodophors. Quick, easy, and convenient to use in general surgery, plastic surgery, orthopaedic surgery, neurosurgery, cardiovascular surgery, and radiation therapy.

**Product construction**

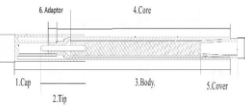Skin Marker  
Product no. 66 x x x x  
Product specification  
Just for reference  
Variation 5%

**Intended use**

Skin marker and selection of the surgical site for diagnostic and interventional radiology, surgery, medical teaching and treatment of skin diseases.

**Directions for use**

1. Sterilize the intact skin with 75% alcohol and iodophors according to standard procedure and let it air dry (about 6 seconds).
2. Test the marker on a small area of intact skin to ensure it is working properly and to check for any allergic reactions.
3. Apply gentle pressure while marking to ensure the dye transfers effectively and the mark is visible.
4. If additional marks are needed, repeat the previous steps as necessary.
5. After use, dispose of the marker safely and properly as medical waste.
6. Marking can be easily removed with a Chlorhexidine Gluconate solution.

**Warnings**

1. Consider the possibility that the patient may be allergic to Gentian Violet. If patient feels uncomfortable while using, stop using and treat appropriately.
2. Radiation sterilized, for single use only, do not reuse or re-sterilize. The reuse may cause cross contamination which leads to microbial infection.
3. Should be used on dry skin. If using on sweating or wet skin, please wipe dry and operate as above Directions for use.
4. Protect the tip during use and cover with cap when finish using.
5. It should be used immediately after opening the package and disposed of safely and properly as medical rubbish after use.
6. Do not use if package or product has been damaged or soiled.
7. DO NOT ingest. In case of accidental ingestion, DO NOT induce vomiting. Rinse mouth thoroughly with water and seek for medical advice.
8. Avoid contact with eyes. In case of accidental contact, flush eyes with water and seek medical advice.

**Contraindication**

Use on mucosa and open wound is not recommended.

**Possible complications**

None

**Storage**

Store in a cool, dry place. Avoid exposure to water, direct sunlight extreme temperature, high humidity or corrosive gases.  
Temperature limit 5°C - 25°C  
Sterile for 3 years. Expiry date : see package

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