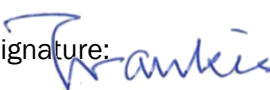
Document Name and Number:

D-730-02 Sterile Disposable Skin Marker IFU (Radiation Sterilization)_V2

Document information, authorship, and approvals			
Author signs to confirm technical content			
Prepared by: Denise Mak	Job title: Quality Assistant	Signature: 	Date: 7/10/2024
Subject matter expert reviewer signs to confirm technical content			
Reviewed by: Frankie Ng	Job title: Managing Director	Signature: 	Date: 7/10/2024

Document history			
Revision No.	Revision Date	Description of change	Author
V1	7/10/2024	Initial Release	Denise Mak
V2	25/11/2024	Addition of Dual Tips	Denise Mak

Sticla Diagonala Stia Marker
Marcador de piel desechable estéril
Marqueur cutané stérile jetable
Pensacelo cutaneo monouso sterile
Marcador de pelo descartável estéril
Sticler Hautmarkiergerät
Sticli hudmarker för engångsbruk
Sticli Tek Kullanımlık Cilt İşaretleme

1. English
2. Español
3. Français
4. Italian
5. Portuguese
6. Deutsch
7. Svenska
8. Türkisch

Primary structure / Estructura primaria / Structure primaire
Struttura primaria / Estructura primária / primäre Struktur
Primär struktur / Birincil yapı

Dongguan Concord Medical Products Ltd.
Floor 4, Block C, Yue Neng Ind Park
Xia Bian Village, Houjie, Dongguan
China 523960
Tel : +86-769-88919066 Fax : 88919050
www.concord-medical.com

MD **MD** mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen

Document No. D-730-02
Version 2nd Release
Issue Date 25/11/2024

Symbols and label descriptions
Símbolos y descripciones de etiquetas
Symboles und Eikettenbeschreibungen

REF Item Number
número de artículo
numéro d'article
codice articolo
número de item
Katalognummer
Artikelnummer
Ürün numarası

LOT lot number
numero de lote
numéro de lot
numero di lotto
número de lote
Chargennummer
Partinummer
Lot Number

MD Date of manufacture
Fecha de manufactura
Date de fabrication
Data di produzione
Data de fabricação
Herstellungsdatum
Tililverkningsdatum
Üretim tarihi

MD Use by date
Utilizar por fecha
Utiliser par date
Usare entro la data
Usar por data
Verwendbar bis
Använd senast datum
Tärlhe Ötre Kulan

MD Keep away from sunlight
Mantener alejado de la luz solar
Tenir à l'écart du soleil
Tenere lontano dalla luce solare
Mantenha longe da luz solar
Vor Sonnenlicht schützen
Håll borta från solljus
Güneş ışılarından uzak tutun

MD Do not use if package is damaged and consult instructions for use
No utilizar si el paquete está dañado y consultar las instrucciones de uso.
Ne pas utiliser si l'emballage est endommagé et consulter les instructions d'utilisation
Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso
Não use se a embalagem estiver danificada e consulte as instruções de uso
Nicht verwenden, wenn die Verpackung beschädigt ist, und Gebrauchsanweisung beachten
Använd inte om förpackningen är skadad och se bruksanvisningen
Paket hasırlysa kullanmayın ve kullanılm talimatlarına bakın.

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ENGLISH Sterile Disposable Skin Marker

Description

Skin marker is a medical pen used for marking on the patient's skin before or during surgical procedures


Specifications

#	Models	Brand	Line width	With ruler	Label
1	663021	/	Regular Tip (1mm)	No	No
2	663022	/	Regular Tip (1mm)	Yes	No
3	663023	Concord	Regular Tip (1mm)	Yes	6
4	663024	/	Regular Tip (1mm)	Yes	9
5	663025	/	Regular Tip (1mm)	Yes	6
6	663121	/	Fine Tip (0.5mm)	No	No
7	663122	/	Fine Tip (0.5mm)	Yes	No
8	663123	Concord	Fine Tip (0.5mm)	Yes	6
9	663124	/	Fine Tip (0.5mm)	Yes	9
10	663125	/	Fine Tip (0.5mm)	Yes	6
11	663221	/	Dual Tips	No	No
12	663222	/	Dual Tips	Yes	No
13	663223	/	Dual Tips	Yes	6
14	663224	/	Dual Tips	Yes	9

Characteristics

Skin marker is used in marking and selecting the surgical sites before or during surgical procedures. Long-lasting scrub-resistant, practically odourless ink, contains primarily gentian violet, which is non-toxic, non-allergenic and non-irritating. Single-use, radiation sterilized product that will not come off easily during the disinfection procedure with 75% alcohol and iodophors. Quick, easy, and convenient to use in general surgery, plastic surgery, orthopaedic surgery, neurosurgery, cardiovascular surgery, and radiation therapy.

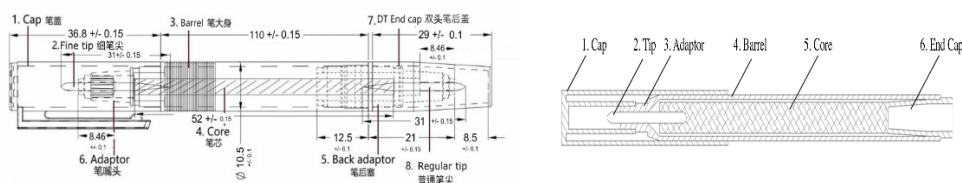Product construction

Intended use

Skin marker and selection of the surgical site for diagnostic and interventional radiology, surgery, medical teaching and treatment of skin diseases.

Directions for use

1. Sterilize the intact skin with 75% alcohol and iodophors according to standard procedure and let it air dry (about 6 seconds).
2. Test the marker on a small area of intact skin to ensure it is working properly and to check for any allergic reactions.
3. Apply gentle pressure while marking to ensure the dye transfers effectively and the mark is visible.

4. If additional marks are needed, repeat the previous steps as necessary.
5. After use, dispose of the marker safely and properly as medical waste.
6. Marking can be easily removed with a Chlorhexidine Gluconate solution.

Warnings

1. Consider the possibility that the patient may be allergic to Gentian Violet. If patient feels uncomfortable while using, stop using and treat appropriately.
2. Radiation sterilized, for single use only, do not reuse or re-sterilize. The reuse may cause cross contamination which leads to microbial infection.
3. Should be used on dry skin. If using on sweating or wet skin, please wipe dry and operate as above Directions for use.
4. Protect the tip during use and cover with cap when finish using.
5. It should be used immediately after opening the package and disposed of safely and properly as medical rubbish after use.
6. Do not use if package or product has been damaged or soiled.
7. DO NOT ingest. In case of accidental ingestion, DO NOT induce vomiting. Rinse mouth thoroughly with water and seek for medical advice.
8. Avoid contact with eyes. In case of accidental contact, flush eyes with water and seek medical advice.

Contraindication

Use on mucosa and open wound is not recommended.

Possible complications

None

Storage

Store in a cool, dry place. Avoid exposure to water, direct sunlight extreme temperature, high humidity or corrosive gases.

Temperature limit 5°C - 25°C

Sterile for 3 years. Expiry date : see package

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