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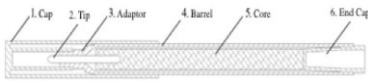
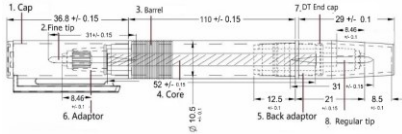
EC REP mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen

Document No. D-730-40
Version 2nd Release
Issue Date 2025/01/20

ENGLISH Sterile Disposable Skin Marker

Description

Skin marker is a medical pen used for marking on the patient's skin before or during surgical procedures.

Specifications

#	Model	Brand	Line width	With ruler	Label
1	663021	/	Regular tip (1mm)	No	No
2	663022	/	Regular tip (1mm)	Yes	No
3	663023	Concord	Regular tip (1mm)	Yes	6
4	663024	/	Regular tip (1mm)	Yes	9
5	663025	/	Regular tip (1mm)	Yes	6
6	663121	/	Fine Tip (0.5mm)	No	No
7	663122	/	Fine Tip (0.5mm)	Yes	No
8	663123	Concord	Fine Tip (0.5mm)	Yes	6
9	663124	/	Fine Tip (0.5mm)	Yes	9
10	663125	/	Fine Tip (0.5mm)	Yes	6
11	663221	/	Dual Tips	No	No
12	663222	/	Dual Tips	Yes	No
13	663223	/	Dual Tips	Yes	6
14	663224	/	Dual Tips	Yes	9

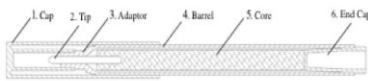
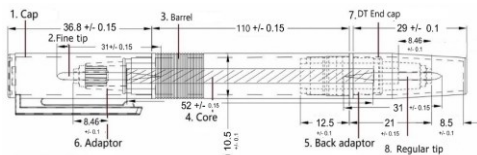
Characteristics

Skin marker is used in marking and selecting the surgical sites before or during surgical procedures.

Long-lasting scrub-resistant, practically odourless ink, contains primarily gentian violet.

Single-use, radiation sterilized product that will not come off easily during the disinfection procedure with 75% alcohol and iodophors. Quick, easy, and convenient to use in general surgery, plastic surgery, orthopaedic surgery, neurosurgery, cardiovascular surgery, and radiation therapy.

Product construction

Intended use

Skin marker and selection of the surgical site for diagnostic and interventional radiology, surgery, medical teaching and treatment of skin diseases.

Directions for use

1. Sterilize the intact skin with 75% alcohol and iodophors according to standard procedure and let it air dry (about 6 seconds).
2. Test the marker on a small area of intact skin to ensure it is working properly and to check for any allergic reactions.
3. Apply gentle pressure while marking to ensure the dye transfers effectively and the mark is visible.
4. If additional marks are needed, repeat the previous steps as necessary.
5. After use, dispose of the marker safely and properly as medical waste.
6. Marking can be easily removed with a Chlorhexidine Gluconate solution.

Warnings

1. Consider the possibility that the patient may be allergic to Gentian Violet. If patient feels uncomfortable while using, stop using and treat appropriately.
2. Radiation sterilized, for single use only, do not reuse or re-sterilize. The reuse may cause cross contamination which leads to microbial infection.
3. Should be used on dry skin. If using on sweating or wet skin, please wipe dry and operate as above Directions for use.
4. Protect the tip during use and cover with cap when finish using.
5. It should be used immediately after opening the package and disposed of safely and properly as medical rubbish after use.
6. Do not use if package or product has been damaged or soiled.
7. DO NOT ingest. In case of accidental ingestion, DO NOT induce vomiting. Rinse mouth thoroughly with water and seek for medical advice.
8. Avoid contact with eyes. In case of accidental contact, flush eyes with water and seek medical advice.

Contraindication

Use on mucosa and open wound is not recommended.

Possible complications

None

Storage

Store in a cool, dry place. Avoid exposure to water, direct sunlight extreme temperature, high humidity or corrosive gases.
Temperature limit 5°C - 25°C
Sterile for 3 years. Expiry date : see package

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